STERYLAB PRODUCT RANGE

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 The risk of post-spinal hemicrania, can be reduced not only by using fine needles, but also through a different geometry of the needle point.

► The two cutting edges of standard anaesthesia needles make a "V" incision of the *dura mater*.

► "Pencil-point" needles don't have any cutting edge. The hole through *dura mater* can therefore be performed more through dilatation than cutting.

► Using the most advanced manufacturing techniques, STERYLAB has developed a new pencil-point needle. Small but significant differences in tip geometry facilitate ideal needle placement.

► This allows:
- positive "dura-click" recognition
- clear differentiation of the different tissue layers
- minimal puncture depth and avoidance of nerve lesions

► Using our long experience of special needle manufacturing has allowed us to produce DURA-JECT®, with optimised and advanced tip geometry, increased flexibility and point stability, using the best materials available.

► DURA-JECT® allows:
- ease of use
- optimal liquor flow
- low injection pressure
- safety and high success ratio

► DURA-JECT® is available in a wide range of sizes to fulfil the needs of the operator.

► INTRODUCER NEEDLE
On request a specific introducer needle is available (see table of sizes).

► Sterile, single use: reduction of infection hazard.

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STERYLAB S.r.l. • VIA MAGENTA 77/6 • 20017 RHO (MILANO) • ITALY